PERISPHERA®

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PERISPHERA®

Personal perineal probe for re-education by means of Electrical Stimulation & EMG biofeedback

 Carefully read this document before using the device!

Recommendations

This probe should be used only with devices complying with the following specifications:

- Biphasic pulse current, zero net D.C. average (galvanic-free)
- Assume an impedance of 500 Ohms
- maximum current: 50 mA_{RMS}
- maximum voltage: +/- 80 V (160 volts peak to peak)

CONTRAINDICATIONS 

Do not use this probe for electrostimulation in presence of confirmed or suspected: bladder, vaginal or anal infection, colorectal or genitourinary cancer, pregnancy, patient with a cardiac pacemaker (consult the doctor who prescribed the pacemaker), ureterovesical reflux and hypotonic bladder.

WARNING!

Do not immerse the probe in water, or in a disinfectant liquid. - Do not allow the connector to become wet - Do not connect to a high frequency surgical apparatus (risk of burns at the site of the electrodes). - Do not disassemble nor rotate the parts of the probe. - The long term effects of stimulation have not been established - Follow the protocol. - DO NOT sterilize! - Do not autoclave!

INSTRUCTIONS FOR USE

The probe is strictly personal, it therefore does not have to be sterilized. Sterilization in liquid can alter the metalized parts. It must be cleaned after each use with mild soap and water. When used for the first time, the tamper-proof packet must result sealed. The probe is not sterilised however it is disinfected, before the packaging. Before use it for the first time, it is recommended to rinse it under tap water, in order to remove eventual disinfectant residuals.

The probe must be kept away from sulphur or any products containing sulphur.

The probe can be used for about 60 treatments.

Warranty

The probe carries a 2 year warranty against manufacturing defects, provided that it is used according to instructions. Return any defective probe in the original package.

For operating hints visit www.perisphera.com

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